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GB Unit and controls

- ➊ Sensor cover ➋ Sensor ➌ Button for Forehead measurement mode
➍ Indikator - LED ➎ Verlichtete LCD-weergave ➏ SET-toets
➐ Battery compartment (on backside) ➑ Memory-button ➒ Beeper ➓ Date ➔ Time
➕ Symbol for continuous temperature measurement ➖ Measurement unit (°C or °F)
➗ Measurement result ➙ Icon for activated sound ➚ Battery exchange symbol ➛ Memory display
➜ Surface measurement mode ➝ Ear measurement mode ➞ Forehead measurement mode

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This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

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Protection classification against solid objects and water

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Operation - Getting started

Removing / inserting batteries: For correct function, the included 2 batteries 1.5V (LR03,AAA) need to be inserted. When the battery exchange symbol  appears in the display after a longer time of usage, the batteries should be replaced soonest. When the battery exchange symbol  appears together with «Lo» or nothing at all appears in the display , the batteries need to be replaced immediately.

Removal: Remove the lid of the battery compartment  and take out the batteries.

Insertion: Put in two new 1.5V-batteries of type LR03 / AAA. Make sure, the batteries are the right way around (see the diagram in the battery compartment ). Put the battery compartment lid back.

BATTERY SAFETY INFORMATION

- Do not disassemble batteries!
- Clean the battery and device contacts if necessary before putting in the batteries!
- Remove discharged batteries from the device immediately!
- Increased risk of leakage, avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of these parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed seek medical attention immediately!
- Replace all of the batteries simultaneously!
- Only place with batteries of the same type, never use different types of batteries together or used batteries with new ones!
- Insert the batteries correctly, observing the polarity!
- Keep the battery compartment well sealed!
- Remove the batteries from the device if it is not going to be used for an extended period!
- Keep batteries out of children's reach!
- Do not attempt to recharge these batteries! There is a danger of explosion!
- Do not short circuit! There is a danger of explosion!
- Do not throw into a fire! There is a danger of explosion!
- Keep unused batteries in their packaging away from metal objects in order to prevent short circuiting!
- Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

Before use

Accustomise the thermometer to the temperature of the room: For an accurate reading of a patient's temperature, their body must first acclimatise to the temperature of the room where the measurement will be made. The same applies to the Infrared multifunctional thermometer **TM 750**. When taken somewhere else, it automatically adjusts itself to the new environment. The thermometer takes a certain amount of time to equalise its own temperature to that of the room. This depends on the difference between the two temperatures and can take more than 30 minutes. Precise readings can only be taken once this process is completed.

Setting the unit of measure, the time and the date:

When the thermometer is used for the first time or after a battery change, the unit of measure, the time and the date must first be set. If you want to return to the setting mode later on, press and hold the SET-button  for approx. 5 seconds, when device is switched on. After the self-test (where all the display symbols are shown in succession in the display ) two short beeps can be heard. Now you adjust the unit of measure (°C or °F) with the SET-button . Press the Memory-button  to confirm the setting. Use the same procedure (SET-button  to adjust a value and Memory-button  to confirm) for setting the year, month, day, hour and minute. After the last press on the Memory-button  the display  shows «OFF» and then the device will switch off automatically.

Information on measuring temperature

- The sensor  of the thermometer must be kept clean and intact at all times in order to guarantee that the measurements are accurate.
- Ear wax affects the measuring accuracy. If necessary, clean your ear and the sensor tip of the thermometer before measuring in order to avoid erroneous measurements.
- Before taking a measurement on the forehead, clean the area and remove any dirt, sweat or cosmetics before moving the sensor to the measuring position as otherwise erroneous results could be measured.
- Do not measure your temperature immediately after eating, training or bathing. Wait for half an hour.
- Do not take successive measurements at intervals of less than one minute. If you have to take your temperature at shorter intervals, the results may fluctuate slightly.
- The human body temperature fluctuates within a natural range. There is no such thing as a standard body temperature. Measure and record body temperature on normal days. This will allow you to detect a fever more easily.
- In order to make an accurate measurement in the ear, the sensor must be directed at the eardrum and be inserted far enough into the ear canal.
- Some people will have different readings for the right and left ears. Always measure the same person in the same ear when measuring temperature changes.
- The measurements can also be influenced by your sleeping position. If someone has slept on a particular ear for a considerable period, the temperature will be higher than normal in this ear. In this case, measure in the other ear or wait for a few minutes before measuring.

Measuring the temperature at your forehead

- NOTE - Forehead temperatures should only be taken for checking purposes, since the skin temperature at the forehead is easily affected by outside influences, meaning that the measurement can be falsified.**

- Switch on the thermometer by shortly pressing the button for forehead measurement mode . The device performs a self-test and emits two short beeps. Optionally, you now have the possibility to temporarily deactivate the sound output of the device until it is switched off. To do this, briefly press the SET button ; the symbol  disappears.
- In order to measure the body temperature on the forehead correctly, the sensor cover  needs to be put on the sensor.
- Place the measuring head with the sensor (and the put on sensor cover) on a temple and press the button for forehead measurement mode . The symbol for forehead measurement mode  appears in the display. Slowly move the thermometer smoothly over the forehead (button still pressed down) to the other temple. Release the button for forehead measurement mode .
- After the temperature has been determined, the device emits a signal tone. The result will be shown in the display .
- If the measurement result is 38.0°C (100.4°F) or higher, the indicator LED  lights up in red (= fever alarm). In all other cases the indicator LED  lights up in green.
- The measurement result will automatically be saved. After some seconds without pressing any button the device will switch off automatically.

Measuring the temperature in your ear

- To measure the temperature in your ear, first remove the sensor cover .
- Switch on the thermometer by shortly pressing the button for ear measurement mode . The device performs a self-test and emits two short beeps. Optionally, you now have the possibility to temporarily deactivate the sound output of the device until it is switched off. To do this, briefly press the SET button ; the symbol  disappears.
- Stretch your ear canal by pulling your ear backwards and upwards and carefully insert the sensor into the ear. For children under the age of 1 year, the sensor should be inserted into the ear canal, while the child is lying flat (e.g. on a bed) with the head slightly turned, so that the ear points upward. Press and hold the button for ear measurement mode  for approx. 1 second. The symbol for ear measurement mode  appears in the display.
- Release the button. After the measurement process is complete, you will hear a beep and the body temperature will be displayed in the display .
- If the measurement result is 38.0°C (100.4°F) or higher, the indicator LED  lights up in red (= fever alarm). In all other cases the indicator LED  lights up in green.
- The measurement result will automatically be saved. After some seconds without pressing any button the device will switch off automatically.

Measuring the temperature of an object

- Switch on the thermometer by shortly pressing button  or button . The device performs a self-test and emits two short beeps. Optionally, you now have the possibility to temporarily deactivate the sound output of the device until it is switched off. To do this, briefly press the SET button ; the symbol  disappears.
- Press and hold the set  & memory  buttons at the same time for at least 3 seconds. The symbol for the surface temperature measurement mode  appears in the display .
- To begin the measurement, point the sensor  at the middle of the object (resp. of the liquid) at a distance of approx. 3 cm. Press the button for forehead measurement mode  or the button for ear measurement mode . If the object has water, dust or dirt on it, remove this first so that the reading is as accurate as possible.
- Release the button. After the measurement process is complete, you will hear a beep and the measured temperature will be shown in the display .
- The measurement result will automatically be saved. Again press and hold the set  & memory  buttons at the same time for at least 3 seconds to return to body temperature measurement mode. After some seconds without pressing any button the device will switch off automatically.

Recalling stored data

The device can store up to 30 results. You can call up your stored data with a short press of the memory button . Firstly, the memory slot number 1 (the most recently saved value) will appear in the display , together with the temperature reading and the symbol for the measurement mode (ear or forehead measurement mode or surface temperature mode). With a further press of the memory button , you can call up the result stored in memory slot "2". A total of 30 memory slots are available and these will be displayed in order if you continue to press the memory button . If a memory slot is empty, the symbol "-" will appear next to it in the display.

- NOTE - In case all memory slots are used, a new measurement result will be saved by deleting the oldest slot (slot no. 30) automatically.**

Error messages

Display «Hi» : The measured temperature is too high. Hi appears in the display if the reading is above 42°C / 107.6°F (body mode) or 100°C / 212°F (object mode).

Display «Lo» : The measured temperature is too low. Lo appears in the display if the reading is below 35°C / 95°F (body mode) or 0°C / 32°F (object mode).

Display «Err» : The ambient temperature is not in the tolerance range of 15°C - 35°C (59°F - 95°F).

Cleaning and maintenance

- The sensor  is the most delicate part of the device. Protect it from dirt and damage. Clean the sensor with a cotton bud dipped in warm water or 70% isopropyl alcohol.
- Do not immerse the thermometer in water or any other liquid. **It is not waterproof.**
- To clean the thermometer, use a soft cloth slightly dampened with mild soapy water or 70% isopropyl alcohol. Do not let water get into the unit. Only use the unit again when completely dry.
- Never use aggressive cleaning agents, thinners, benzine or tough brushes.
- Take care not to scratch the lens of the sensor, the display or the housing.
- Do not use or store the unit in an environment where the ambient temperature or humidity is too high or too low (see operating conditions in the "Technical specifications" section), in direct sunshine, in the vicinity of static electricity or in a dusty environment.
- Remove the batteries if you do not wish to use the unit for an extended period of time. Otherwise there is a danger that the batteries may leak.
- If the thermometer is used as described in these instructions, no regular maintenance or calibration is required.

Disposal

-  This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Please remove the batteries before disposing of the device / unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop. Consult your municipal authority or your supplier for information about disposal.

Technical specifications

Name and model:	medisana Infrared multifunctional thermometer TM 750
Display system:	Digital display (LCD)
Power supply:	3 V ==, 2 batteries 1.5 V (type LR03, AAA)
Measuring range:	Body temperature measurement mode: 34°C - 43°C (93.2°F - 109.4°F)
Accuracy of measurement: (laboratory readings)	Surface temperature measurement mode: 0°C - 100°C (32°F - 212°F) [under operating conditions] Body temperature measurement mode: In the measuring range of 35°C - 42°C (95°F - 107.6°F): ± 0.2°C (0.4°F); Object mode: ± 2°C (4°F) or ±5%
Display resolution:	0.1°C (0.1°F)
Memory capacity:	up to 30 readings
Operating conditions:	15°C - 35°C (59°F - 95°F), rel. humidity up to 95% (not condensing)
Storage / transport conditions:	25°C - +55°C (+13°F - +131°F), rel. humidity up to 95% (not condensing)
Dimensions:	approx. 134.3 x 42.4 x 47.6 mm
Weight:	approx. 65 g without batteries
Article No.:	76140
EAN Code:	40 15588 76140 9

The CE-mark of this device refers to the EC-directive 93/42/EEC MDD. The device complies with ISO 80601-2-56. The device has been designed for domestic use. A metrological inspection should be carried out annually if the device is used on a commercial basis. There is a charge for this inspection which can be carried out by a competent authority or authorised maintenance service providers in accordance with the Medical Devices Operator Ordinance. Electromagnetic compatibility: The device complies with the EN 60601-1/-1-2 standards for electromagnetic compatibility. Please check the supplementary sheet for details on this measurement data.

In accordance with our policy of continual product improvement, we reserve the right to make technical and optical changes without notice.
--

The current version of this instruction manual can be found under www.medisana.com

Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

- The warranty period for **medisana** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
- Defects in material or workmanship will be removed free of charge within the warranty period.
- Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
- The following is excluded under the warranty:
 - All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
 - All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - Accessories which are subject to normal wear and tear.
- Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

	medisana GmbH Jagenbergstr. 19 41468 NEUSS, GERMANY	The service centre address is shown on the attached leaflet.
---	--	--

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